





Balloon G-Tube

Instructions for use


Connecting and disconnecting the G-Tubes luer port to enteral syringes and feeding administration sets

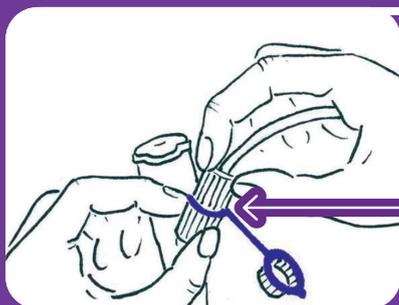


- When connecting or disconnecting a feeding set, syringe or cap, hold the G-Tube luer connection firmly between the thumb and forefinger of one hand
- Rotate the accessory connection only one quarter turn clockwise to close and one quarter turn anti-clockwise to open

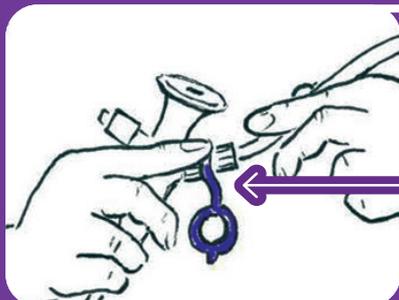

- DO NOT hold the G-Tube as per image as the luer connector on the tube could become loose

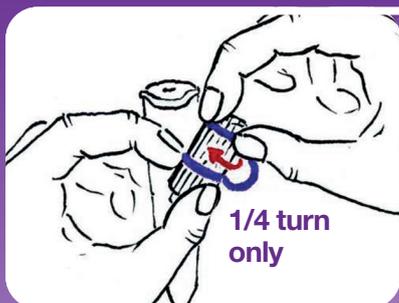

- The connector requires only a one-quarter turn to lock or unlock
- Do not overtighten
- Use of clamps to open and close this port are not recommended as they can cause damage to the connector

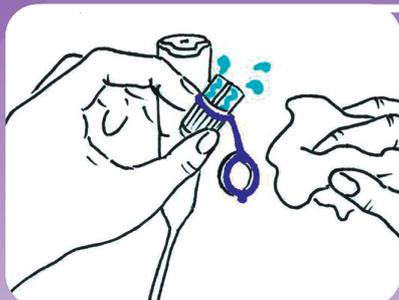

- When priming the feeding administration set and syringes try to avoid spilling medication or feed into the luer lock port itself
- If any spillage does occur this needs to be cleaned before closing the luer connector
- Close the tethered cap with a one-quarter turn clockwise



Scan for more information & order codes



Scan the QR to contact your local Area Business Manager:

Email: info@gbukenteral.com
Main: +44 (0)1757 282 945



GBUKEnteral
Part of the GBUK Group