

# INSTRUCTION FOR USE

## Safety Blood Collection Set & Holder



### Blood Collection


1. Remove device from blister pack using aseptic technique, if necessary, and connect it to the desired collection system.
2. Carefully remove needle cap from the safety winged needle.
3. Prepare the venipuncture site and perform venipuncture according to standard approved technique. Check flashback to confirm successful venipuncture.
4. Secure the winged needle with tape if necessary.
5. Collect blood by placing vacuum tube into the tube holder, replace tube if needed. Please refer to the relevant local guidelines



### Safety Mechanism Activation


#### A. Using one-handed procedure:

- A1. After completion place gauze over venipuncture site. With the other hand, withdraw the blood collection set by holding the grid of the yellow safety shield with index finger and thumb.
- A2. Continue holding the grid of the yellow safety shield with index finger and thumb, while securely grasping the tubing on the back using the other three fingers.
- A3. Slide forward the yellow safety shield with index finger and thumb, until the needle is fully covered by the safety shield and an audible click is heard – this indicates the safety mechanism is activated.

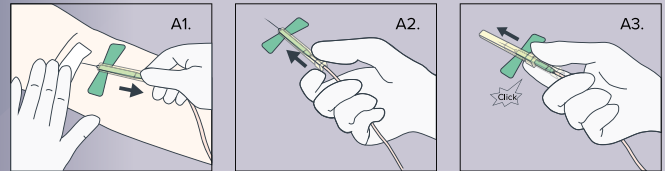


#### B. Using two-handed procedure:

- B1. After completion place gauze over venipuncture site. With the other hand, withdraw the blood collection set by holding the grid of the yellow safety shield with index finger and thumb.
- B2. Hold the tubing with index finger and thumb of the opposite hand.
- B3. Slide the yellow safety shield forward, until the needle is fully covered by the safety shield and an audible click is heard – this indicates the safety mechanism is activated.

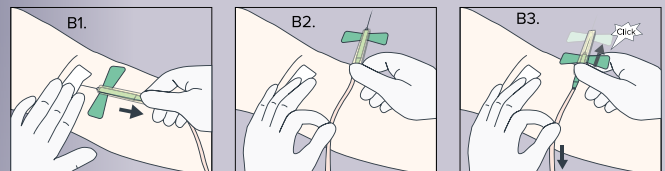

#### C. In-vein activation using one-handed procedure:

- C1. After completion place gauze over venipuncture site. With the other hand, withdraw the blood collection set by holding the grid of the yellow safety shield with index finger and thumb.
- C2. Maintain the angle of injection and continue holding the grid of the yellow safety shield with index finger and thumb, while securely grasping the tubing on the back using the other three fingers. Start retracting needle from vein.
- C3. While retracting needle, slide forward the yellow safety shield with index finger and thumb until the needle is fully retracted and covered by the safety shield. When an audible click is heard the safety mechanism is activated.

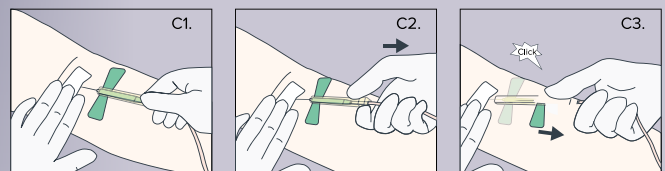

Scan for more information & order codes

[www.gbukhealthcare.com](http://www.gbukhealthcare.com)



Scan the QR to contact your local Area Business Manager:

Email: [info@gbukhealthcare.com](mailto:info@gbukhealthcare.com)  
Main: +44 (0)1757 288 587



**GBUKHealthcare**  
Part of the GBUK Group